



Chocolate Agar Base

RDM-CAB-01

Principle

Chocolate Agar Base, composed of proteose peptone that provides the organism with nitrogenous nutrients, amino acids, and other elements essential for the growth of the organisms. Dextrose is carbon and energy source. Phosphates buffer the medium. Sodium chloride maintains osmotic balance. Agar is a solidifying agent.

The selectivity is provided by addition of glutamine, coenzymes, cocarboxylase, hematin and growth factors required by fastidious organisms. Thayer-Martin Media is a chocolate agar supplemented with vancomycin, nystatin, and colistin to inhibit the normal flora, including nonpathogenic *Neisseria* for the selective isolation of *N. gonorrhoeae* and *N. meningitidis*.

Use: Recommended for the isolation of *Neisseria gonorrhoeae* from chronic and acute cases of gonococcal infections.

Contents*

Ingredients

	Gram/Litre
Protease Peptone	20.00
Dextrose	0.50
Sodium Chloride	5.00
Disodium hydrogen Phosphate	5.00
Agar	15.00
pH at 25°C	7.2 ±0.2

* Formula adjusted for optimum performance and parameters

Directions: Dissolve 45.50 grams in 495 ml distilled water. Boil to dissolve the medium completely and sterilize by autoclaving at 15 lbs pressure (121 °C) for 15 min, cool it to 42-45 °C. Add 500 ml of sterile hemoglobin solution and mix thoroughly and distribute aseptically in petri plates. Ensure complete solidification and inoculate test sample aseptically.

Sterile 2% hemoglobin solution: Add 10 ml of hemoglobin to distilled water and bring volume to 500 ml. Mix thoroughly and autoclave for 15 min at 15lbs pressure at 21°C. Cool at 45°C -50°C.

The media can also be enriched with yeast autolysate supplement or vitamin growth supplement reconstituted. Mix well before pouring.

Specimens types analyzed

Pharmaceutical samples, clinical and non-clinical samples etc.

Precautions to be taken

These microbial media are intended for the in-vitro use only. All the handling, experiments, storage, and discarding should be performed with the help of skilled and knowledgeable technicians and as per the established guidelines. The material should be disposed only after proper sterilization by autoclaving. Please go through the MSDS of the media to avoid any accidents or in emergency.

Performance and Evaluation

The expected performance of the medium is liable to use as per the direction on the label when stored at optimum conditions and within expiry date.

Quality Control

Appearance	Beige colored free flowing, homogeneous powder
Reaction of 4.55% solution	7.2 ±0.2 at 25 °C
pH	7.00- 7.40
Gelling	Firm comparable with 1.5 % agar gel
Color and clarity of ready medium	Light amber colored opalescent gel (without hemoglobin solution)
Color and clarity of ready medium	Chocolate brown colored opaque gel (with hemoglobin solution)
Growth Promotion properties	Best at ≤ 100 CFU at 32-37 °C for 18-72 h
Indicative properties	Optimum at ≤ 100 CFU at 32-37 °C for 18-48 h
Negative control	Performed using sterile distilled water

Different Microbial Response

Organism	ATCC	Inoculum	Growth	Recovery	Incubation
<i>Neisseria gonorrhoeae</i>	19424	50-100	Luxurious	70-80%	33-37 °C, 18-48 h

Storage and Shelf Life

Hygroscopic; keep container tightly closed. Store in cool dry place.

Disposal: To avoid the contamination or propagation of any hazardous microbes the used, unusable or modified preparation of this product must be disposed after autoclaving after completion of task.

Reference

1. Atlas, R. M. (2005). *Handbook of media for environmental microbiology*. CRC press.
2. *Difco Manual* (1998). 11th Edition. Difco Laboratories., Division of Becton Dickinson and Company, Sparks, Maryland, USA.
3. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) *Manual of Clinical Microbiology*, 11th Edition. Vol. 1.

Disclaimer

The information contained in the technical data sheet is to the best of our knowledge is accurate and true based on the research and development work carried out by **ReadyMED®**, Chaitanya Agro Biotech, Malkapur, Maharashtra. The products are neither intended for any therapeutic use for animal or human nor for any other *in-vivo* applications. The **ReadyMED®** products are only meant to be used for the laboratory, diagnostic, research, or further manufacturing purpose only. These technical outcomes should not be considered as the warranty of any kind expressed or implied, and no liability is accepted for infringement of any patent.

CHAITANYA AGRO BIOTECH PVT. LTD. An ISO 11134:2014, ISO 13485:2016, ISO 9001:2015 CE , CIN NO.: U24210MH1995PTC095220S,
S. No. 120/2, Laxmi Nagar, Umbarnala Road, Malkapur-443101, Dist.: Buldana (M.S.) India. Customer Care +91-8669083859
rdmsales@chaitanyagroupindia.com, mkt.cabt@chaitanyagroupindia.com, www.chaitanyagroupindia.com